Speed F-Corona™

Speed F-Corona™

Veterinary diagnostic kit

For *in vitro* use only

■ CLINICAL APPLICATION

Cats infected with Feline Coronavirus display a range of clinical signs. Classic oronasal transmission results in enteritis associated with transient or chronic faecal excretion of the virus. Most cats eliminate enteric coronavirus within a few weeks, however some remain asymptomatic excretory carriers and thus a source of contamination for other cats.

In some cases, viral mutation in the gastrointestinal tract can lead to the development of severe systemic disease: Feline Infectious Peritonitis or FIP. FIP is more common in young cats between 4 months to 2 years of age, and less frequently in cats over 5 years old. There are two different clinical presentations of FIP:

- **The wet form**, with ascites, pleural effusion, or pericardial effusion.
- **The dry form**, which results in a general loss of condition (weight loss, depression, and pyrexia), occasionally associated with ocular and/or neurological symptoms.

Detection of anti-Coronavirus antibodies provides an indication of the disease status of a cat and thus predicts potential faecal excretion; it is also a useful parameter, when combined with indicative clinical signs, in the diagnosis of FIP.

■ PRINCIPLE

Speed F-Corona is a rapid qualitative immunoassay, using immunochromatography, for the detection of anti-coronavirus antibodies, in cats.

For each test, one drop of the sample (serum, plasma, anticoagulated whole blood, or effusion) is added to the sample well. In the sample well, the stained particles of the conjugate bind to any anti-coronavirus antibodies present in the sample. The resulting conjugate/antibody complexes migrate along the membrane via capillarity. They are captured by specific coronavirus antigens that are bound to the membrane, forming a pink test band by accumulation of coloured particles. The mixture continues to migrate along the strip to the end of the membrane where the remaining coloured particles form a pink control band, which confirms the validity of the test.

■ PROCEDURE

► FOR EACH TEST YOU WILL NEED:

1 test device, 1 single-use pipette and the reagent dropper bottle.
The reagents should be used at room temperature.

Do not mix reagents from different batches.

1/ ADD THE SAMPLE:

Serum, plasma, anticoagulated whole blood (EDTA or heparin), or effusion: with a single-use pipette held vertically, **add 1 drop** of sample to the sample well.

2/ ADD THE REAGENT:

- Add the reagent immediately after the sample. Hold the dropper bottle in a vertical position and **add 5 drops** of reagent to the sample well.

- If no migration is observed within 2 minutes, **add an extra 2 drops** of reagent to the sample well.

3/ READING AND INTERPRETATION OF RESULTS:

• **READING:**

Wait 15 minutes for migration and read:

- **NEGATIVE TEST:** any test showing only **1 pink band** (control band) is a negative result.

- **POSITIVE TEST:** any test showing **2 distinct pink bands** (test band + control band) indicates a positive result.

The test is positive if a pink test band appears within 10 minutes of migration. Any colour change in the test band, even faint, should be considered as a positive result.

- **If no control band appears, the test is invalid.**

• **INTERPRETATION:**

A positive result indicates previous contact with feline coronavirus. Seropositive cats may excrete the virus in their faeces.

In sick cats, when used in combination other diagnostic tests, Speed F-Corona provides an indication of the likelihood of the cat suffering from FIP. However, towards the end of the disease process, the formation of immune complexes can reduce circulating antibody levels resulting in false negative results.

■ **RECOMMENDATIONS**

• **SHELF LIFE:**

- 24 months at room temperature (from manufacturing date). The expiry date is printed on each box and on each test pouch.
- Store at room temperature, between +2°C to +30°C. Keep the test away from sources of excessive heat or cold.

• **STORAGE OF SAMPLES:**

- Serum, plasma, anticoagulated whole blood or effusion must be used at room temperature.

- Storage within 24 hours: store serum, plasma, anticoagulated whole blood, or effusion at room temperature.
- Storage between 24 to 72 hours: store serum, plasma, or effusion between +2°C and +8°C (refrigerator). Storing whole blood between +2°C and +8°C for several hours may induce haemolysis (not visible to the naked eye), which may interfere with the interpretation of the results.
- Storage beyond 72 hours: store serum, plasma, or effusion at -20°C. Never freeze a whole blood sample.

• **PRECAUTIONS:**

- To ensure correct migration of the sample, place the test device on a horizontal surface.
- To standardize the amount of sample recommended for each test, only use the single-use pipettes supplied in the kit.
- To ensure that the sample and reagent are correctly applied, always hold the pipette or dropper bottle vertically.

• **OTHER RECOMMENDATIONS:**

- The reading window may show a light pink background coloration; this does not affect the quality of the results.
- **Do not mix reagents from different batches.**
- Allow the reagents to come to room temperature before use.
- Use a new single-use pipette for each test. Only use the pipettes supplied in the box.

The above recommendations are only guidelines; no test is 100% accurate at all times and under all conditions. The purpose of this test is to detect Feline anti-coronavirus antibodies to assist practicing vets with the diagnosis of FIP; test results should be interpreted in the light of the patient's history, a full clinical examination, and the results of any further diagnostic tests. The definitive diagnosis remains the veterinarian's prerogative and responsibility.

Bio Veto Test and its distributors cannot be held responsible for the consequences of misuse or misinterpretation of the results of the test.

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